

easymoov6®

Enteral feeding pump
for ambulatory use





Value Life

Tubing installation

Place the silicone loop around the rotor and make sure the cassette is in the right direction.

Push the cassette in the middle of the pins.

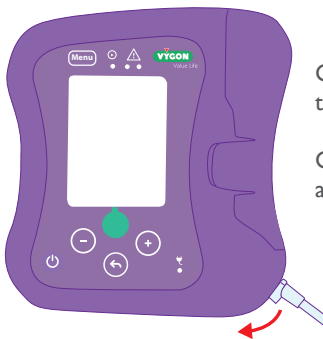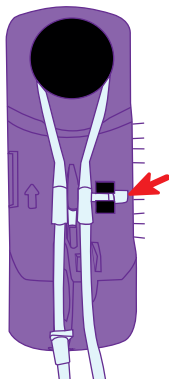

Guide both tubes down, in line with their respective slots.

Close the pump cover, then place the adapter into the designated slot.

Explanation of symbols

Notification symbols



: Indicates that lock mode is activated



: Indicates that the intermittent mode is activated



: Indicates that the keypad is locked



: Indicates sound level



: Indicates battery level

11:09 : Indicates the clock

Symbols in the central part of the screen



: Indicates that feeding is running



: Indicates that the pump is paused



: Flow rate programmed



: Volume programmed


Time Left : Time remaining before the end of feeding

Total Fed: Volume already administered

Tubing priming


Switch on the pump and prime tubing

Switch on the pump by pressing  and prime tubing by pressing . Disconnect the patient and press  to prime.

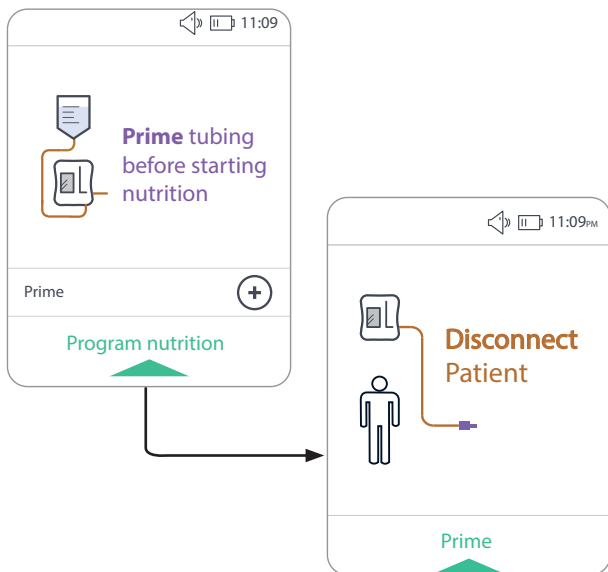

Verify the tubing is fully primed



If necessary, restart priming by pressing  or program the feeding by pressing .

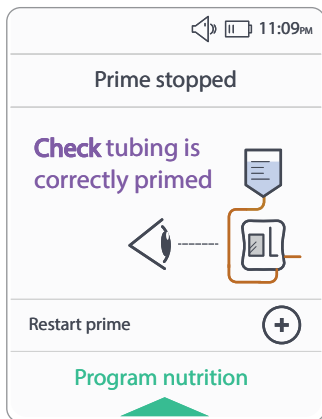

Program the feeding

Adjust the flow rate


Press \ominus and \oplus to set the flow rate (1 to 400ml/h).
Then press \bullet to confirm.

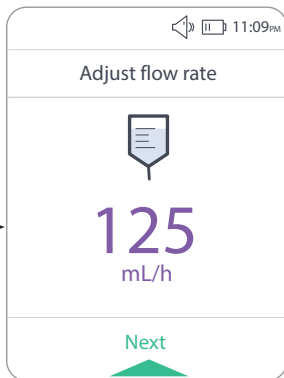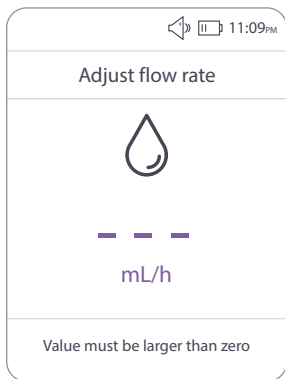

Program the feeding



Adjust the volume

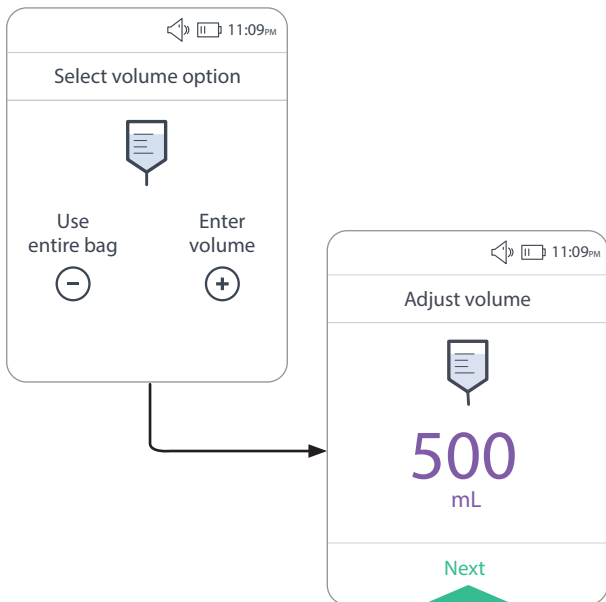

Press \ominus to empty the bag or \oplus to set a volume (1 to 9,999ml).
Then press ● to confirm.






Program the feeding


Feeding running



The feeding is running. Press and hold  to pause the pump. The pause mode allows you to change the flow rate and the volume or to switch the pump off by pressing .


  11:09PM



Running 


 Rate	 Total
125 mL/h	500 mL
Time left	Total Fed
4 h 00 min	0 mL


Hold to pause 

  11:09PM



Paused 

 Rate	 Total
125 mL/h	500 mL
Time left	Total Fed
4 h 00 min	0 mL

Change rate or volume 


Resume nutrition 

Alarms and corrective actions


For each warning message, first silence the alarm by pressing . Then, read the message on the screen and correct the error. Press  to resume feeding.

Alarms

Corrective actions

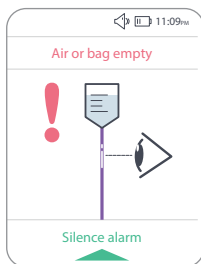

Check the level of the bag and the positioning of the tubing.
Disconnect the tube from the patient and launch a prime.
If the problem persists, remove and replace the upstream part of the tube.

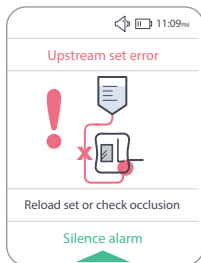


Release upstream occlusion (between pump and bag) and make sure the tubing is correctly positioned.
Make sure the ultrasound sensor is clean.

Alarms and corrective actions


Alarms

Corrective actions

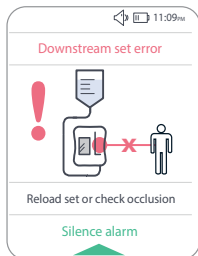


Release downstream occlusion (between pump and patient) and make sure the tubing is correctly positioned.

Make sure that the infrared sensor is clean and that the cassette is not full of feed.

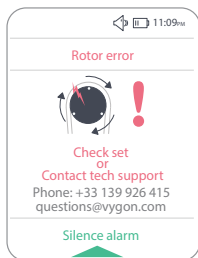

Check the positioning of the tubing around the rotor (black wheel) and that nothing is blocking it.

If the alarm still appears, contact the customer service.



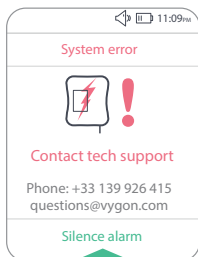


Clean the black pins of the infrared sensor, switch the pump off and back on.

If the message persists, contact the customer service.

Alarms and corrective actions

Alarms

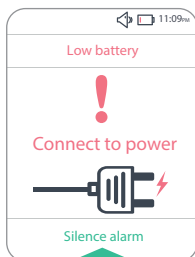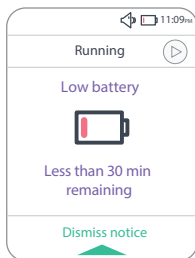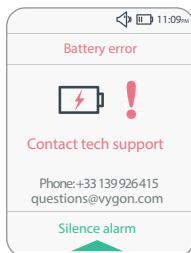

Corrective actions

Contact the customer service.

Low battery, 30 minutes remaining.
Connect the AC adapter to the pump and a power source.

5 minute life battery.
Connect the AC adapter to the pump and a power source.

Technical information

Cleaning instructions:

Before any cleaning operation, unplug the pump from power source. Use a soft cloth moistened with soap and water to clean the pump. The cleaning must be carried out every day or when necessary.

Battery:

Charging time: 2h30
Autonomy: 20h at 125 ml/h

Memory:

Specific values and modes: 24h

For further information:

For further information,
please refer to the instructions for use.



Manufactured by Medwin France
 9 Allée de la vigne Grande
34600 Les Aires-FRANCE

Distributed by :
Vygon
5 rue Adeline - 95440 ECOUEN - FRANCE
Customer service : +33 1 39 92 64 15
Service commercial France +33 1 39 92 63 81
Fax : +33 1 39 90 29 37
www.vygon.com – questions@vygon.com